

Leukosan® Adhesive

Patient Information

Dear Patient,

Your wound has been closed using Leukosan® Adhesive, a barely visible adhesive, requiring no stitches or staples. Leukosan® Adhesive will fall off towards the end of the healing process. To ensure your wound heals quickly and with the best cosmetic result, please follow the instructions below:




You may take quick showers, however, please avoid prolonged soaking or bathing.



Refrain from physical activities that induce sweating or include intense muscle flexing until your skin is completely healed.



After showering, carefully pat dry the wound area and do not rub the adhesive.

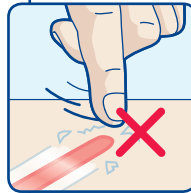

Do not remove Leukosan® Adhesive before healing is complete. Discourage children from picking at the film.



Do not apply creams, lotions or liquid medications onto the adhesive or surrounding area.



If you experience unusual swelling, redness or pain, or if your wound opens, please see your doctor immediately.



Please avoid direct exposure to sunlight and tanning lamps to achieve the best cosmetic result.



If your wound is covered with a bandage, please replace bandage if it gets soaked.

Doctor's address and contact details:

Doctor's comments:

Follow up review:

YES

NO

If yes, Date: _____

BSNmedical

DISTRIBUTED BY:

Smith & Nephew Pty Ltd Healthcare Division

Australia 315 Ferntree Gully Road (PO Box 242), Mount Waverley 3149, Victoria Australia
T 61 3 8540 6777, F 61 3 9544 5086, Customer Service T 13 13 60, F 1800 671 000, www.smith-nephew.com.au/healthcare

New Zealand 621 Rosebank Road Avondale (PO Box 442), Auckland 1140 New Zealand

T 64 9 828 4059, F 64 9 820 2867, Customer Service T 0800 807 663, F 0800 263 222, www.smith-nephew.com/nz